DEUTSCH

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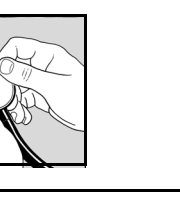
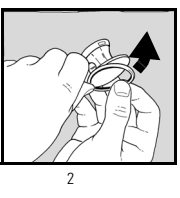
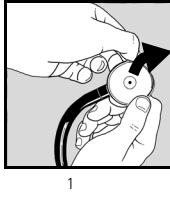
ENGLISH

Designed to enable the medical professional to obtain superior and accurate auscultation.

The Elite[®] Stethoscope was designed after extensive research and testing. The critical design of the tubing, proper weight and sizing of the chestpiece and rotatable binaurals, combine to make the Elite Stethoscope an exceptional instrument for auscultation. The final testing, of course, was on the patient. The Elite Stethoscope will give you the ability to identify the lowest frequencies that the human ear can detect, such as a faint diastolic gallop or rumble, as well as the higher frequencies of the faint early blowing aortic diastolic murmur. Furthermore, with over five decades of experience using stethoscopes, Dr. W. Proctor Harvey has designed the Elite to meet the comfort and portability needs which physicians demand.

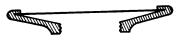
Chestpieces

The Elite Stethoscope offers both a diaphragm and bell, which you select by holding the chestpiece stem in one hand and rotating the chestpiece with the other hand until a click is heard or felt.


Diaphragm

"The workhorse." Excellent for all sounds and murmurs. Best for higher frequencies (such as faint diastolic murmur of aortic regurgitation). Also ideal to identify splitting of sounds, systolic clicks and ejection sounds. 4.85 cm diameter.

Bell

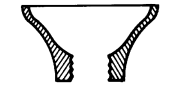
Provides excellent detection and clarity for low and medium frequency heart sounds and murmurs (such as a faint gallop or diastolic rumble). 3.25 cm diameter.





Headset Adjustment

Your new Elite Stethoscope is designed to give you the maximum ability to adjust for your personal comfort and acoustic fit.

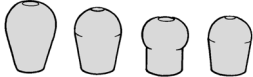1. Select the correct size and texture eartip. For maximum acoustic seal, two sizes of "soft" and "hard" eartips are provided. Choose the size which best seals off your outer ear canal and feels the most comfortable. As a rule the larger size is best for the majority.

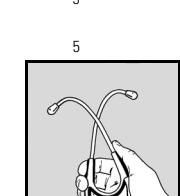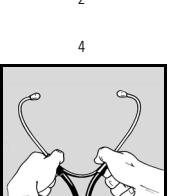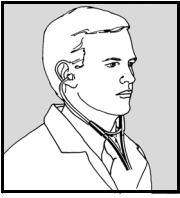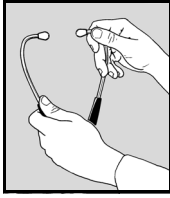2. To adjust the angle of the binaurals, hold the spring in one hand and rotate each binaural to the correct angle.
3. Make sure that the binaural tubes are pointed slightly forward going into the ear canal.
4. To reduce spring tension, grasp each eartube at its bend and gently pull outwards.
5. To increase spring tension, squeeze the eartubes together.

Care and Maintenance

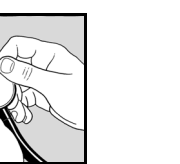
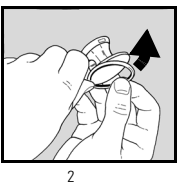
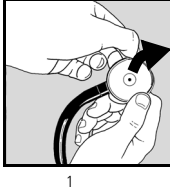
It is important to clean and disinfect your stethoscope to prevent possible cross-contamination.

Eartubes and tubing can be wiped down with alcohol or mild soapy water.

Eartips may be removed for thorough cleaning with alcohol or mild soapy water.

Diaphragm replacement: Under normal use you should rarely need to remove the rim and diaphragm for cleaning purposes. The exterior of the diaphragm can be cleaned with alcohol or mild soapy water. However, if you do need to remove diaphragm, follow the instructions below.

1. Holding the diaphragm side facing you, grip the retaining rim with thumb and index finger and push rim off the edge of the metal chestpiece.
2. To reassemble, align the diaphragm on the chest-piece and push upward.
3. Continue to totally fit the rim onto the chestpiece by aligning the parts in an upward motion until the rim completely snaps on.

Warranty

Your Elite stethoscope comes with a 10-year warranty against parts and manufacturing defects. Should a material or workmanship defect be discovered, Welch Allyn will repair or replace the product at no charge. This warranty does not cover damage in shipment or failure caused by tampering, obvious carelessness or abuse.

Welch Allyn[®]

Elite[®] Stethoscope

Directions for Use

WelchAllyn[®]

Advancing Frontline Care[™]

Material No. 95P411 Rev. E

Welch Allyn USA (Headquarters)
Tel: 1-315-685-4560 or 1-800-535-6663
Fax: 1-315-685-0054

Welch Allyn Ireland (European Customer Service)
Tel: (353) 46 90 67700
Fax: (353) 46 90 67756

Welch Allyn Canada
Tel: 1-905-890-0004
Fax: 1-905-890-0008

Welch Allyn U.K.
Tel: 0-207-365-6780
Fax: 0-207-365-9694

Welch Allyn Italy
Tel: (+39) 02-6968-2425
Fax: (+39) 02-6963-3697

Welch Allyn Australia
Tel: (+61) 2-9638-3000
Fax: (+61) 2-9638-3500

Welch Allyn China
Tel: (+86) 21 6327 9631
Fax: (+86) 21 6327 9632

Welch Allyn Japan
Tel: (+81) 3-219-0071
Fax: (+81) 3-219-0074

Welch Allyn Singapore
Tel: (+65) 291-0882
Fax: (+65) 291-5780

Welch Allyn Germany
Tel: (+49) 7477 9271 70
Fax: (+49) 7477 9271 90

Welch Allyn France
Tel: (+33) 60 09 33 66 or (+33) 55 69 58 49
Fax: (+33) 60 09 67 97

Welch Allyn Spain
Tel: (+34) 917499357
Fax: (+34) 917499361

Welch Allyn South Africa
Tel: (+27) 11 777 7509
Fax: (+27) 11 777 7510

Welch Allyn, Inc.
4341 State Street Road
Skaneateles Falls, NY 13153
Tel. 315.685.4560 Fax 315.685.3361 www.welchallyn.com

The CE mark on this product indicates that it is in conformity with the provisions of Council Directive 93/42/EEC.

European Regulatory Manager, Medical Division, Welch Allyn Ltd. Navan Business Park, Dublin Road, Navan, County Meath, Republic of Ireland
Tel: +353 46 90 67700 Fax: +353 46 90 67756

© 2006 Welch Allyn, Inc. All rights reserved. No one is permitted to reproduce or duplicate, in any form, this manual or any part thereof without permission from Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product that may result from failure to use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual. Welch Allyn[®] is a trademark of Welch Allyn, Inc.

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